

Medication preparation range

Tookan[®] pill crusher

The Tookan[®] pill crusher enables to reduce drugs to **powder**. As it is equipped with an integrated handle which allows to crush all medicines without any efforts, it is suitable for anybody.

Thanks to the storage of bags in the Tookan[®], **no loss neither contamination** are possible with other medicines. Hygiene is therefore completely respected.



Hygienic



Ergonomic

Advantages

- ▶ Easy use
- ▶ Resistant to disinfecting products of hospital environments
- ▶ Crushing without any effort
- ▶ Bags are stored in the crusher

Medication preparation range

Tookan[®] pill crusher

Technical features

Tookan[®] pill crusher

- ▶ Steel base with epoxy coating
- ▶ Aluminium handle
- ▶ Anodised aluminium crushing part
- ▶ Stainless steel axis

Bags for pill crusher

- ▶ Transparent low density polyethylene
- ▶ Thickness: 180/200 µm
- ▶ Environment-friendly



Title	Dimensions (in mm)	Reference
Manual pill crusher Tookan[®]	L. 265 x D. 75 x H. 90	NA00017
Batch of 1000 bags for Tookan[®]	L. 50 x H. 100	NA00011.01

Operating instructions



1 Take a bag in the storage compartment.



2 Insert medicines into the bag and fold it up.



3 Put the drugs into the polyethylene bag and place the bag into the crusher.



4 Press smoothly on the handle with fingertips without releasing the bag.



5 You can take the drugs which are now reduced to powder.